



G11

VARIANT ANALYSIS MODE

AUTOMATED GLYCOHEMOGLOBIN ANALYZER



START UP



TOSOH G11, if you prefer: **User-friendliness**

- Colour touch screen
- Power on timer
- Automated warming up

"The TOSOH G11 has an automated start up function. When I enter the lab, the instrument has already completed the warming up and is in stand-by mode, ready for me to start."



DAILY CHECK



TOSOH G11, if you want: **Security**

- Reagent lot traceability
- Expiry date traceability
- Handheld barcode reader

"The daily check only requires checking the reagents and replace when necessary. Replacement is very easy and requires minimal time. By using the handheld barcode reader, we have full traceability of the reagents used."



QUALITY CONTROLS




TOSOH G11, if you desire: **Certainty**

- Excellent precision
- Only need 10 µL
- Can be frozen

"When the controls are analysed and in range, we can move on with patient samples. We can also aliquot the controls and freeze them. This is very convenient and cost saving. Good quality controls means good quality patient results, which is our priority."

RESULT VALIDATION

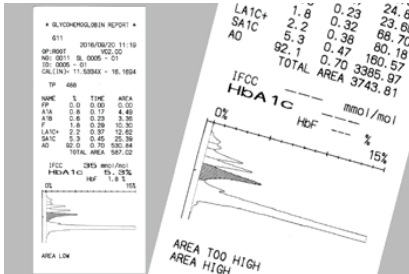

TOSOH G11, if you wish: **Confidence**

- Customisable flags
- Adjustable flag levels
- TP monitoring

"The Tosoh G11 has relieved result validation for me. All normal results are automatically validated and we only need to manually check the abnormal ones. I can also choose to suppress the results that trigger a critical flag, in order to prevent incorrect reporting to our customers."

POWER OFF



TOSOH G11, if you request: **Simplicity**

- Automated stop
- Automated wash
- Power off timer

"What I find user friendly is the rack rotation switch. In this way the analyser knows when the last sample has been analysed and will go directly in wash mode. Afterwards there is a customisable timer to power off the analyser."

FILTER CHANGE



TOSOH G11, if you seek **Protection**

- User-friendly Filter Unit
- Filter capacity 600 Tests
- Ensures High Quality results

"Filter change is done in 2 seconds. Simply open the unit and replace the filter. It is a single action and replacing it, has so many benefits. My column will last longer and it ensures good quality results."

COLUMN CHANGE

TOSOH G11, you deserve: **Peace of Mind**

- Change is done in seconds
- Finger tight connections
- Good quality column



"The column change is very easy so we don't lose precious time. By means of the TP shown on the chromatograms, we know when it is time to replace the column and we are always sure that we report high quality results. This is what gives us peace of mind."

MOGADISHU OFFICE

+252 61 5000491
+252 1 656503

HARGEISA OFFICE

+252 63 4440909
+252 2 512422

GAROWE OFFICE

+252 90 7742425
+252 90 7662918

DJIBOUTI OFFICE

+253 77 021504
+253 77 189595